

XXL REHAB®

SHOWER COMMUNE

We have aimed at designing the best shower commode on the market. It is designed for easy access, independence, safety and on the caregiver terms.

The inside width of the chair is 61 or 71 cm, but using a 5 cm high PU foam seat and 7 cm free space to the armrests allows the body mass to fall down, instead of fading out to the sides. The impossible "wide inside but small outside" is obtained. This allows the XXL-Rehab Shower Commode to pass through nearly all modern doorways.

The footrest is always the weak point when it comes to bariatric products. Normally the producer makes a short cut here, by using a standard solution. We did not only design the footrest to carry 100 kg each, we did more!

The footrest is also side/width adjustable, so access to intimate hygiene is possible and people with the fat-tissue placed on the inside of their leg can sit with legs apart (up to 110 cm) and still have foot support. The brand new footrest is even nicely covered with PU foam.

One of the unique new features is a split toilet seat, a long open hole: 44 cm and 22 cm at the widest point. This to enable a correct use of the chair, it eases access for caregivers and avoids incidents where material ends up in places it wasn't meant to end up!

The ability for bariatric users to reach far back to reach the armrest is usually not present. Therefore we have made the armrests with a length of 64 cm and let it stick out 7 cm from the seat base, ready to welcome and support the hands of mobile users. It enables the user to get into the chair by him/herself. Another option is the butterfly armrests. Butterfly armrests add more than 10 cm to the seating width.

The space between backrest and seat has a clearance of 32 cm. This gives space for a possible bulbous gluteal region, but also enables the caregivers to reach for the sling and position the user better.

- Long armrests provide easy access
- Extra seat width with butterfly armrests



Product video
Scan the code and see
the product in use

STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED EXCLUSIVELY FOR BARIATRIC USERS!

SPECIFICATIONS

Model 1 Model 2

Maximum weight capacity is	325 kg	
Seat width with standard armrests	61 cm	71 cm
Total outer width with standard armrests	68 cm	78 cm
Seat depth	56 cm	
Seat height	55 cm	
Backrest	32-52 cm	
Seat to footrest	40-42-44-46-48 cm	
Width of footrest	58-110 cm	
Pushbar from floor	100 cm	
Width between armrests	54 cm	
Height of backrest	48 cm	
Height of footrests	40-48 cm	
Distance from floor to seat underside	44.5 cm	
Total length	117 cm	
Total weight	27 kg	30 kg
Wheels (2 wheels with individual locks)	125 mm	
Seat	Pressure moulded PUR foam	
Frame	Powder-lacquered chromium molybdenum steel	
ISO - classification	ISO 09 12 03	

ORDERING

Model	Seat width cm	Seat depth cm	Order No
XXL-Rehab Shower commode incl. standard armrests, footrests and commode bucket	61	56	0150-061-000
XXL-Rehab Shower commode incl. standard armrests, footrests and commode bucket	71	56	0150-071-000
XXL-Rehab Shower commode incl. butterfly armrests, footrests and commode bucket	61 + 14	56	0151-061-000
XXL-Rehab Shower commode incl. butterfly armrests, footrests and commode bucket	71 + 14	56	0151-071-000
Accessories			
Butterfly armrests (add 14 cm to seat width)			0150-099-012
Commode bucket			0140-099-001
Footrest right			0150-099-008
Footrest left			0150-099-009
Calf support / amputation support right			0156-099-030
Calf support / amputation support left			0156-099-031
Shower seat			0141-099-010



Calf support / amputation support



Standard armrests



Side/width adjustable footrests



*Butterfly armrests
add 14 cm to seat width*



Soft shower seat